


DEEP SHOOT AA is a multi-functional cocktail containing essential Anti-Aging agents for complete rejuvenation. Key active ingredients such as Glutathione, Peptides, Growth factors and Vitamin B complex are replenished to maintain youthful skin condition and reduce the appearance of aging skin.

Glutathione · Peptides · Growth factors
Vitamin B complex · PDRN

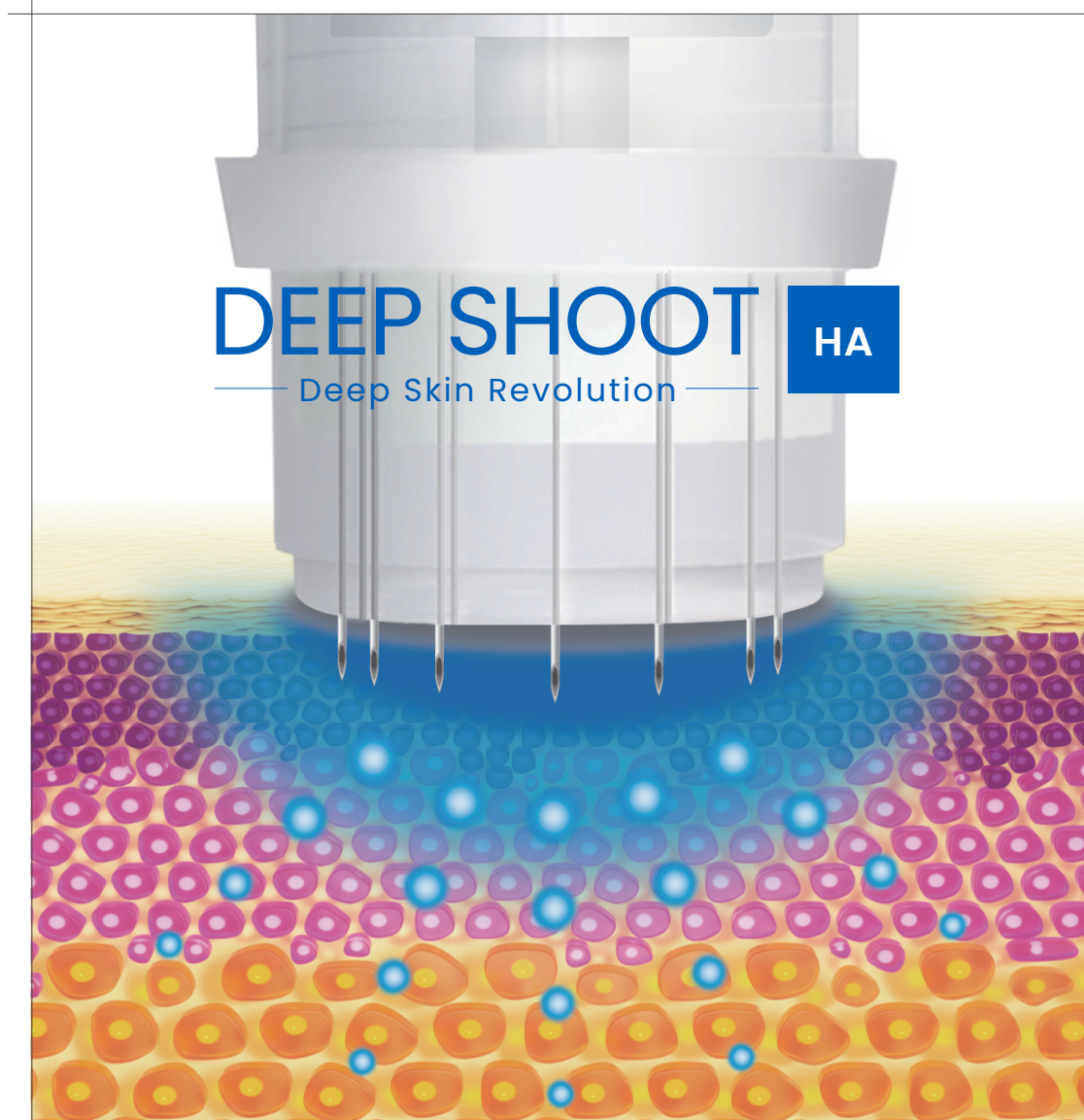



DEEP SHOOT HA is a Hyaluronic Acid provider that replenishes and retains moisture. Three variants of low, medium and high molecular weights promote deeper and stronger action ever for skin hydration.

Hyaluronic Acid-triple action




1 AROUND THE EYES

- Proceed stamping from inside to outside in fan-shaped directions.

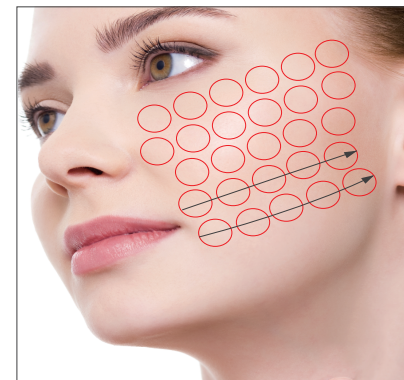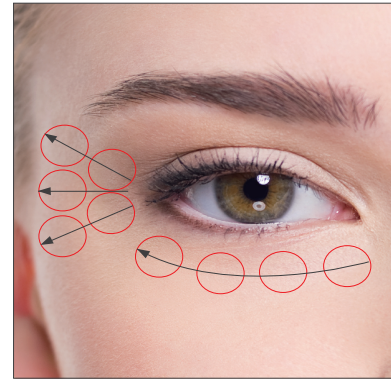


2 CHEEKS

- Proceed stamping into the 45° direction outwards.

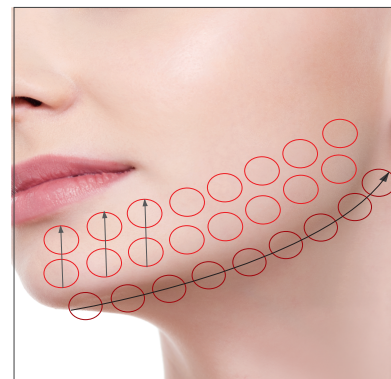


3 JAWLINE

- Proceed stamping from the bottom of the chin into the upright direction along the jawline.
- Proceed stamping from under the chin area into the 45° direction outwards along the jawline.

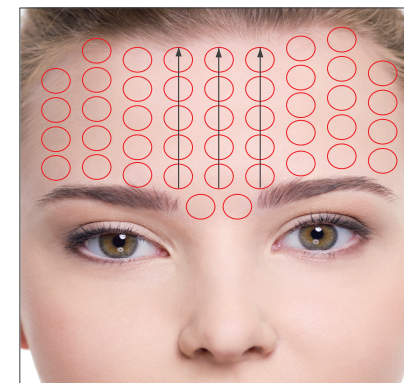


4 FOREHEAD

- Proceed stamping from the bottom line of the forehead, including the middle of eyebrows and temples, into the upright direction.

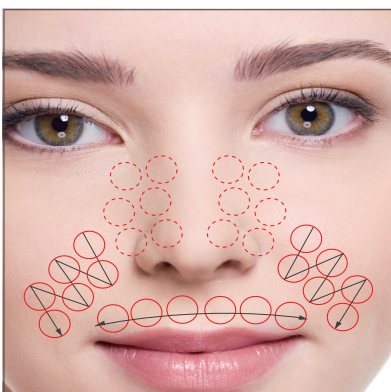



5 NOSE, NASOLABIAL FOLD & PHILTRUM

- Proceed stamping in a zig-zag way along the nasolabial folding line.
- Stamp above the lip line.
- Apply light tapping around the nose.

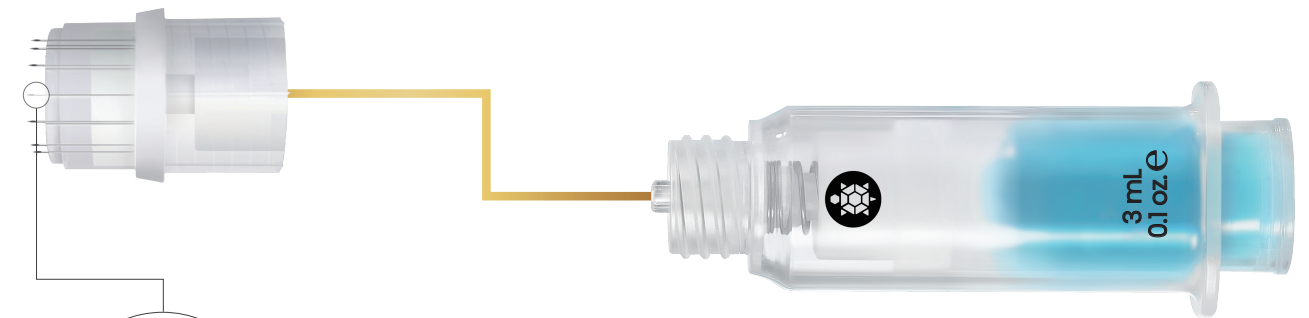

TURTLEPIN[®] Airless Syringe

- User-friendly ergonomic design
- Equal amount of diffusion (0.06cc / 1 stamp)
- New technique for deeper superficial treatment
- No regurgitation or leakage

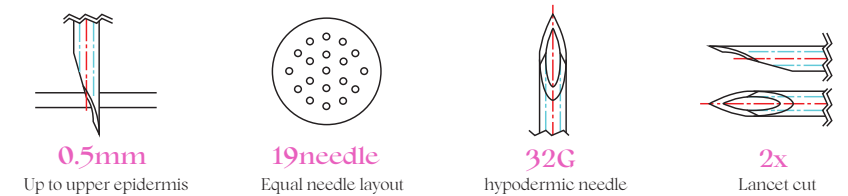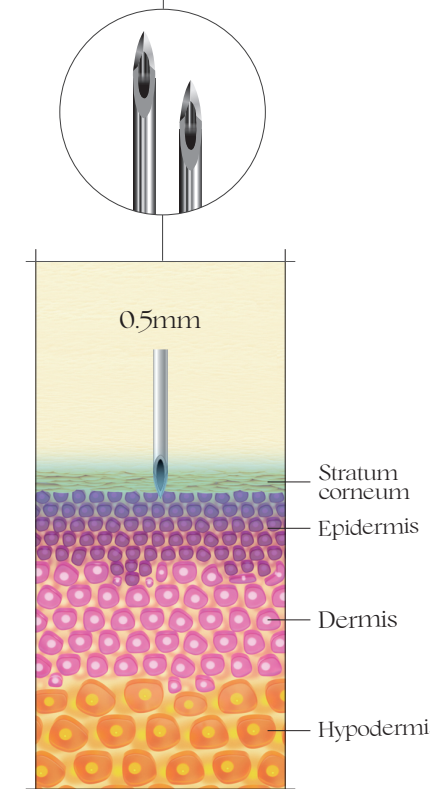

- TURTLEPIN[®] technology enhances the absorption rate of solution by up to 30 times compared to the normal topical application.¹⁾
- It also maximizes the efficacy of transdermal delivery by increasing the absorption rate upto 15times compared to the conventional MTS device using acupuncture pins.^{2) 3)}
- Short recovery time & minimum pain with no down time .

TIPS

- Deeper lines and wrinkles (eg. nasolabial, forehead, between eyebrows, neck etc.) can be covered by stamping from the starting point of the line into a zig-zag way along the line.
- Stretch out the treatment area before stamping at a perpendicular (90°) angle.
- Ready and Shoot! Stay for 2~3 seconds before moving into the next spot.
- Be careful not to slip on the skin surface.
- Tap around the nose (instead of stamping) and treat delicate areas such as around the eyes and lips with caution.

