

SUPER FICIAL

Pink Aging – HL


PROCEDURE GUIDELINE




1 Androgenetic Alopecia (M-shaped pattern)

step 1. Proceed stamping along the hairline starting from the temporal point to the middle of the forehead.

step 2. Divide the session into left and right from the middle part line. Proceed stamping in the indicated direction.

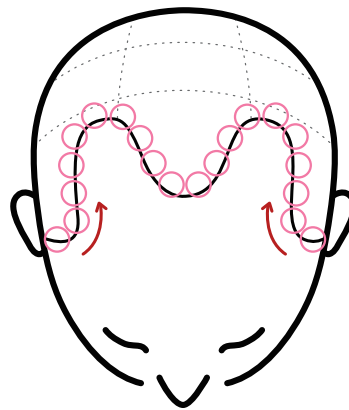

step 1

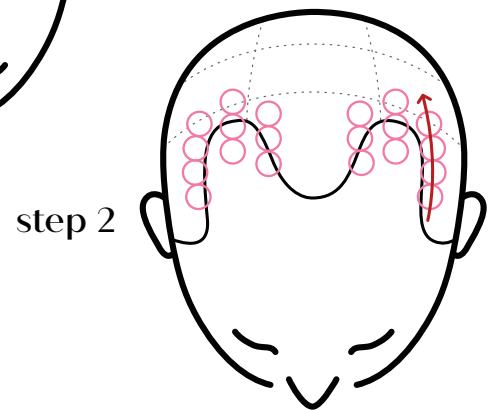


step 2


2 Androgenetic Alopecia (O-shaped pattern)

step 1. Divide the target area vertically into multiple sessions. Proceed stamping in the indicated direction.

step 2. Divide the target area horizontally into multiple sessions. Proceed stamping in the indicated direction.

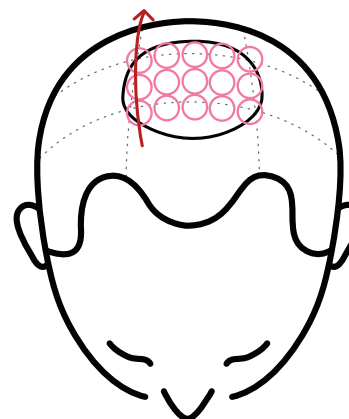

step 1

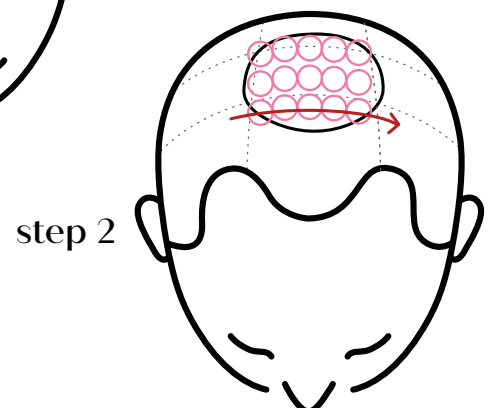


step 2

3 Hair loss around Parts

step 1. Proceed stamping along the part line.



step 2. Create new parts on each side and proceed stamping in the indicated direction.

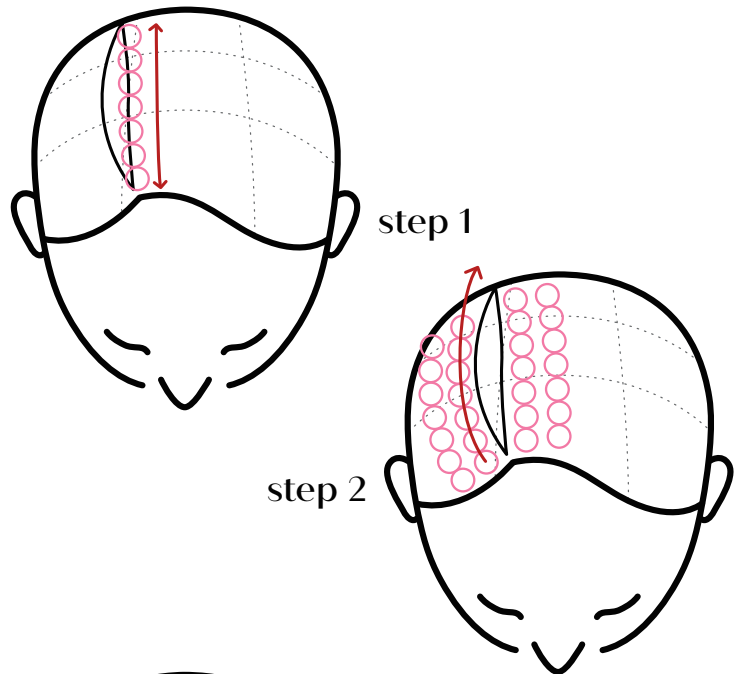

4 Alopecia Areata

step 1. Divide the target area into two directions (horizontal & vertical) and proceed stamping in the indicated direction.

step 2. Create five different parts (one in the middle and two on each side) and proceed stamping to cover the whole scalp area.

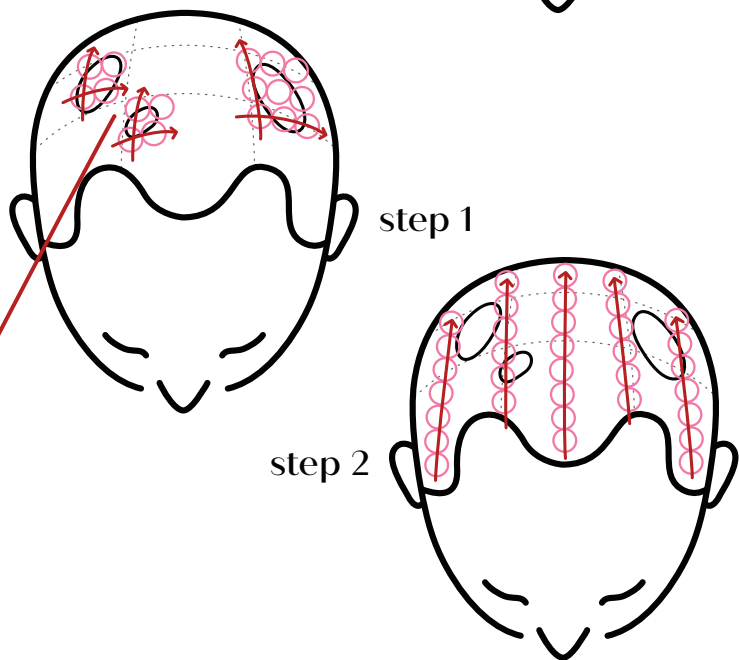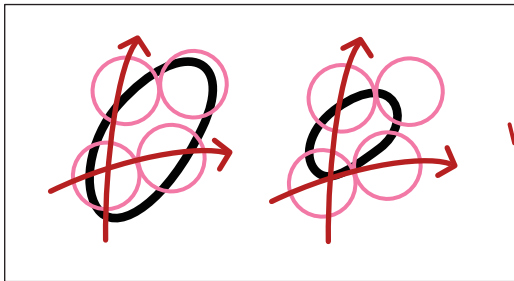

Tips

- Use a tail comb to part hair and/or to divide into different sections for the procedure.
- Hold the part with your hand to secure enough space for the procedure.
- Make sure the stamping is done in the closest contact with the scalp.
- Stamp at a perpendicular (90°) angle.
- Be careful not to slip on the scalp surface.



JMBIOTECH Corporation Limited.
HQ : 42819 Dae-gu, 08513 Seoul, Korea.
EU : OBELIS s.a., 1030 Brussels, Belgium.
RIBESKIN©2022 www.ribeskin.com

For more information

www.ribeskin.com

@ribeskinmed

ribeskin official