

GORGEOUS
GOURI

A new type of Injectable for
lifting the entire Face



About Company



We are supplying Innovative Medical Aesthetic products in Korea specialized in Highpolymer technology.

There is always trend in market and it changes over time.

Company shall satisfy it by recognizing trend.

We, however, do not stop by it.

We lead Trend and provide Brand New Technology

Core Value

01 Dermatology

We are based on Dermatology to achieve Essential Skin Care

02 R&D Base: Leading-Edge High polymer Technology

The World's 1st and Unique Fully Solubilized Biopolymer Technology

03 Innovation

Always try to offer new value, better one to customers

04 Trust

We have pride in products as offering them in the best condition.

We have pride in our clients as putting people first in the best mutual collaboration

About Company

BUSINESS SECTOR

Manufacturing Highpolymer ingredient,
professional cosmetics and medical device



CEO : JAEWON YU



DATE OF INCORPORATION : September 2012



WWW.DEXLEVO.COM



\$430 Million *IPO Valuation(E)

HEADQUARTER

2F, 704, Yeongdong-daero,
Gangnam-gu, Seoul, Korea



R&D / Manufacturing

5F, 632 Seobusaet-gil,
Geumcheon-gu, Seoul, Korea



About DEXLEVO – Certification



CE




ISO 13485




ISO 22716



Authorized manufacturer of Medical Device certificate

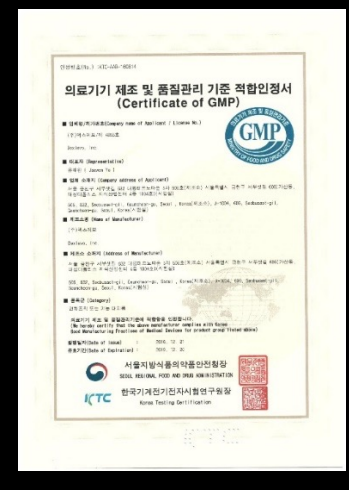


KGMP

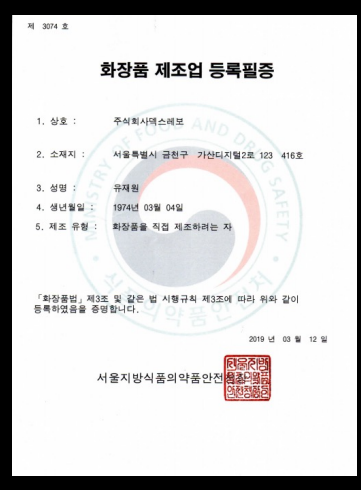


Cosmetic Manufacturer Certificate

Pipeline

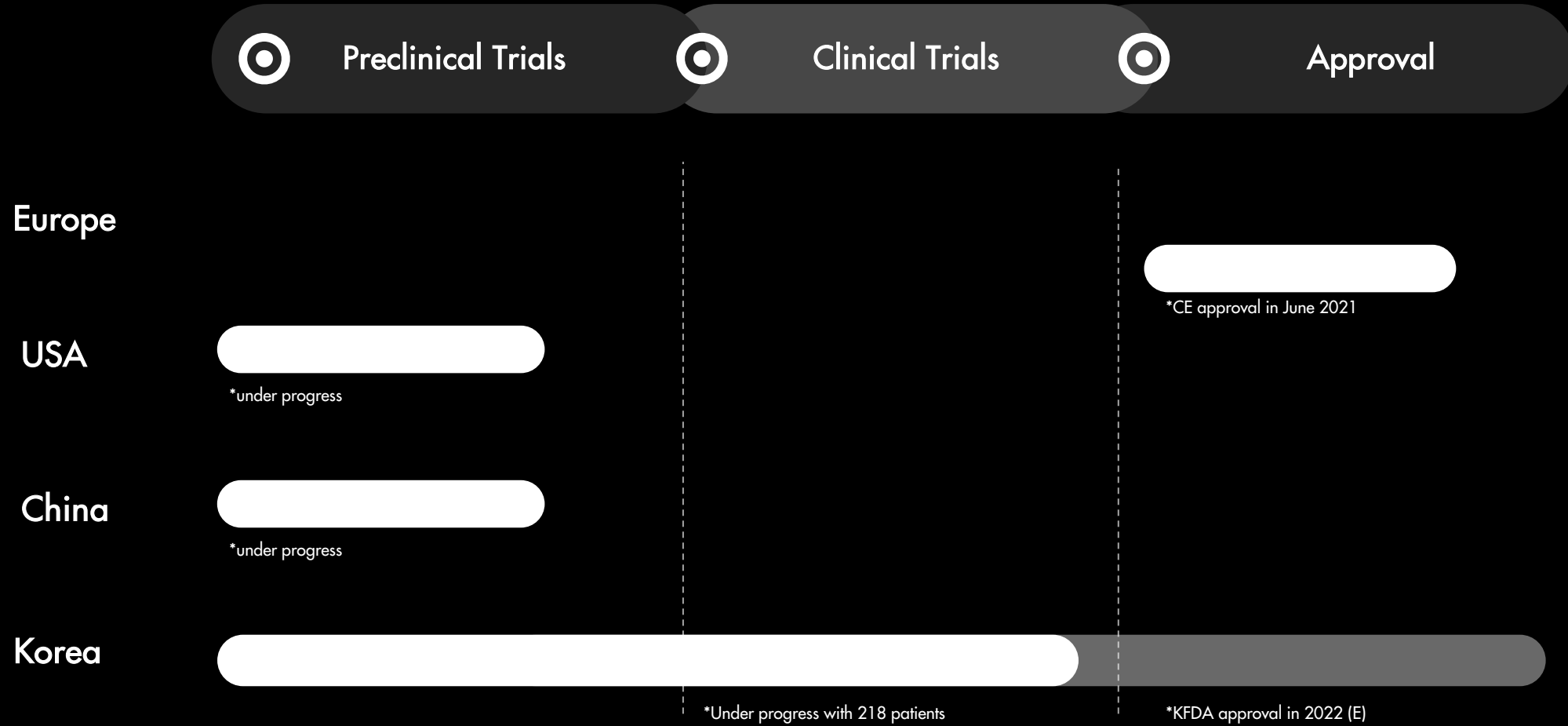

Organization Chart

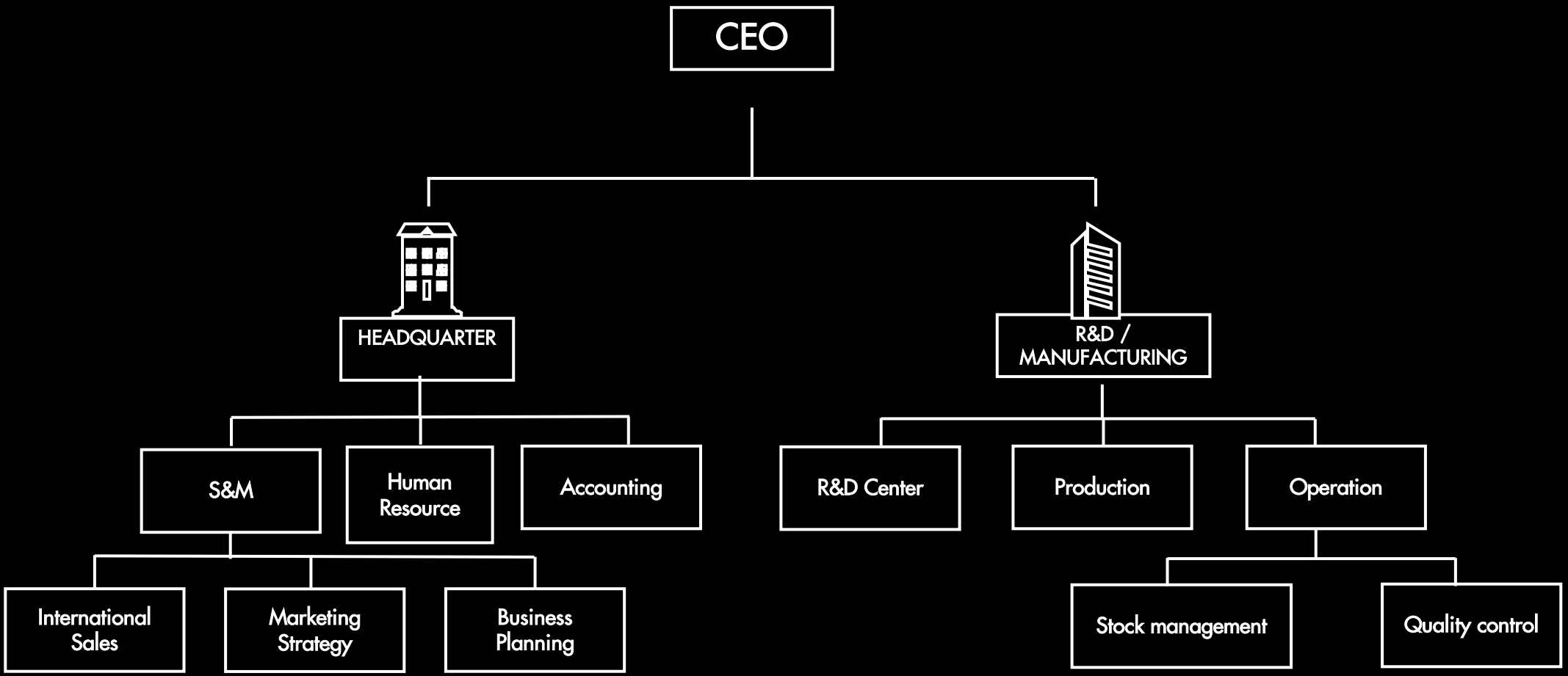

CESABP Technology

by DEXLEVO INC.


DEXLEVO's Collagenesis-Enabled Solubilized Active and Biodegradable Polymer Technology is a unique technology that enables regeneration of natural skin collagen without the use of microparticles.

Features of Key **GOURI**

- Premium Collagen Stimulator
- Fully solubilized liquid form of Polycaprolactone (PCL)



NEO-COLLAGENESIS

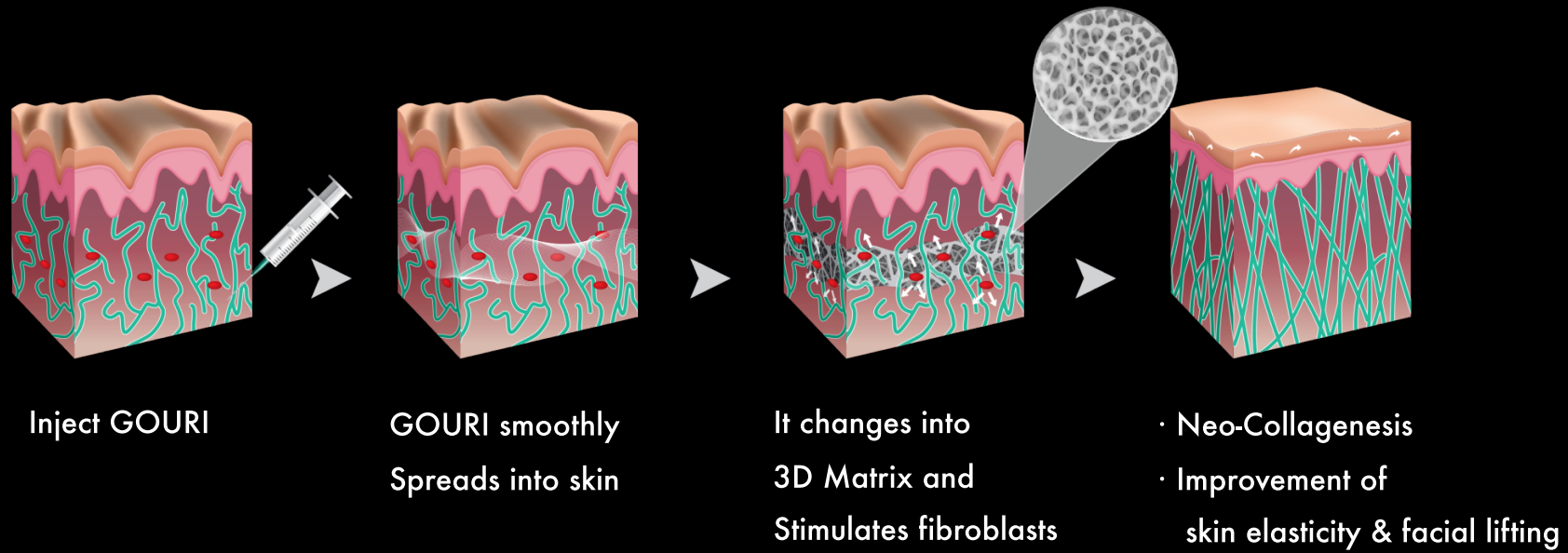

Liquid Form PCL



GOURI naturally spreads around the whole face.

- No Need Multiple Injection Points
- No Need to shape Application Part
- Soft Massage is Enough After Application



● GOURI : Inject 0.2cc * 10 points

HOW TO INJECT GOURI

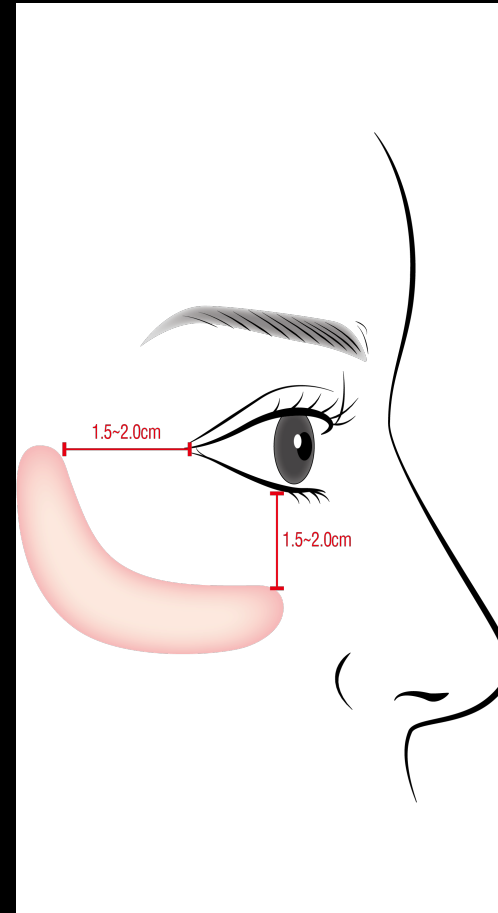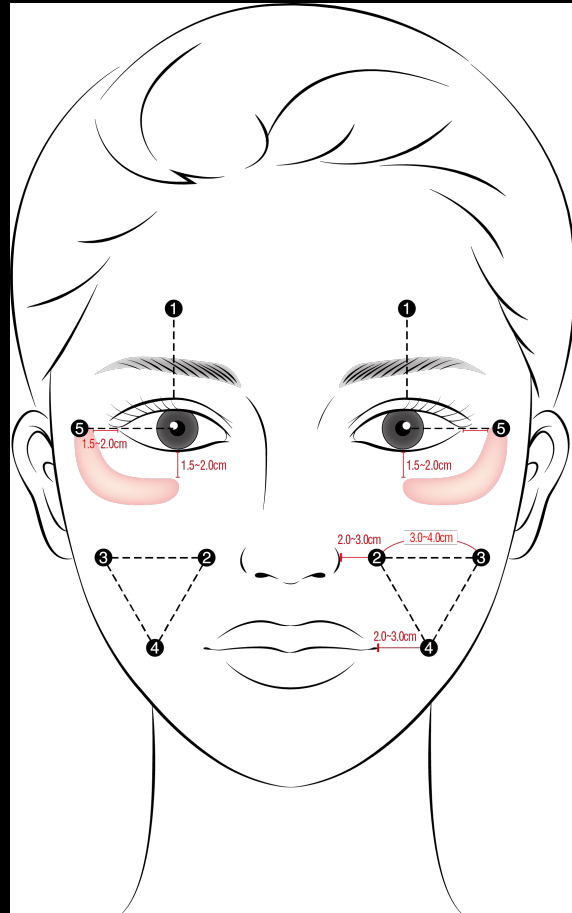


GENERAL Protocol

0.2cc per point

For the best results,
3 sessions are recommended at 4 week interval.

After the 3 sessions, apply additional 1~2 treatments
at 3-6 month interval, depending on patient's condition.

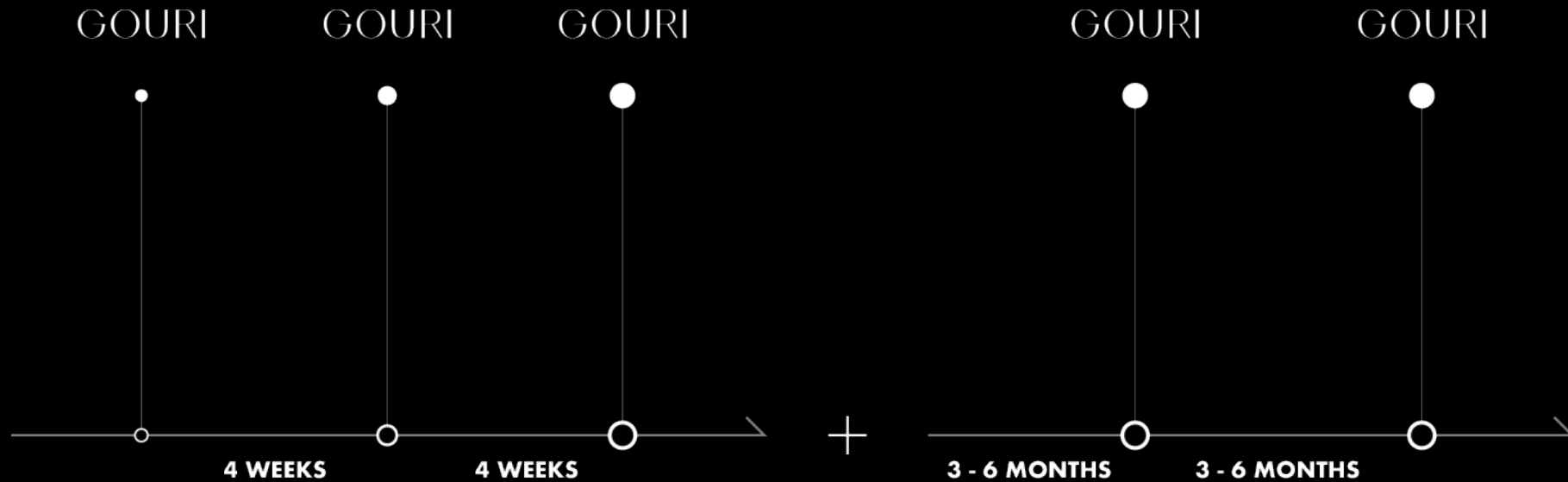

BEFORE & AFTER



Around Eyes



BEFORE TREATMENT

AFTER TREATMENT



BEFORE TREATMENT

AFTER TREATMENT

2 sessions at 4 weeks interval

Around Eyes



BEFORE TREATMENT

AFTER TREATMENT



BEFORE TREATMENT

AFTER TREATMENT

1 session after 12 weeks

BEFORE & AFTER



Full Face



4 sessions at 4 weeks interval

Full Face



4 sessions at 4 weeks interval

GOURI Polycaprolactone
Injectable Implant

DEXLEVO

DEXLEVO INC.

505, Daeryung Techno Town 5-cha, 632,
Seobusaet-gil, Geumcheon-gu, Seoul, Korea

EC REP CMC Medical Devices S.L.
C/ Horacio Lener

GORGEOUS

GOURI

+82-2-6952-8770

info@dexlevo.com

www.gorgeousgouri.com

DEXLEVO INC.